Aesculap® Electrosurgical Instruments and Devices

Aesculap Surgical Technologies

- Table of contents
- Inhaltsverzeichnis
- Sommaire
- Indice
- Contenido

3 - 16	<ul style="list-style-type: none"> — Units and Accessories — Geräte und Zubehör — Appareils et Accessoires — Aparatos y Accesorios — Apparecchiature ed Accessori
17 - 34	<ul style="list-style-type: none"> — Monopolar equipment for open surgery — Monopolares Equipment für die offene Chirurgie — Équipement monopolaire pour la chirurgie ouverte — Equipamiento monopolar para la cirugía abierta — Attrezzatura monopolare per chirurgia a cielo aperto
35 - 70	<ul style="list-style-type: none"> — Bipolar equipment – Coagulation forceps — Bipolares Equipment – Koagulationspinzetten — Équipement bipolaire – Pincés à coagulation — Equipamiento bipolar – Pinzas de coagulación — Attrezzatura bipolare – Pinze per coagulazione
71 - 74	<ul style="list-style-type: none"> — Neutral electrodes — Neutralelektroden — Électrodes neutres — Electrodo neutros — Elettrodi neutri
75 - 82	<ul style="list-style-type: none"> — Cables and Adapter — Kabel und Adapter — Câbles et adaptateurs — Cable y adaptador — Cavi ed adattatori
83 - 94	<ul style="list-style-type: none"> — Electrosurgical Endoscopy — Elektrochirurgie in der Endoskopie — Électrochirurgie en endoscopie — Electrocirugía en la endoscopia — Elettrochirurgia in endoscopia
95 - 98	<ul style="list-style-type: none"> — Numerical Index — Nummernverzeichnis — Index alpha numérique — Índice por referencias — Indice

- Units and Accessories
- Geräte und Zubehör
- Appareils et Accessoires
- Aparatos y Accesorios
- Apparecchiature ed Accessori

GN640

Nelson deluxe – Electrosurgery the innovative way

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– Electrosurgical unit featuring 2 monopolar and 2 bipolar outputs.

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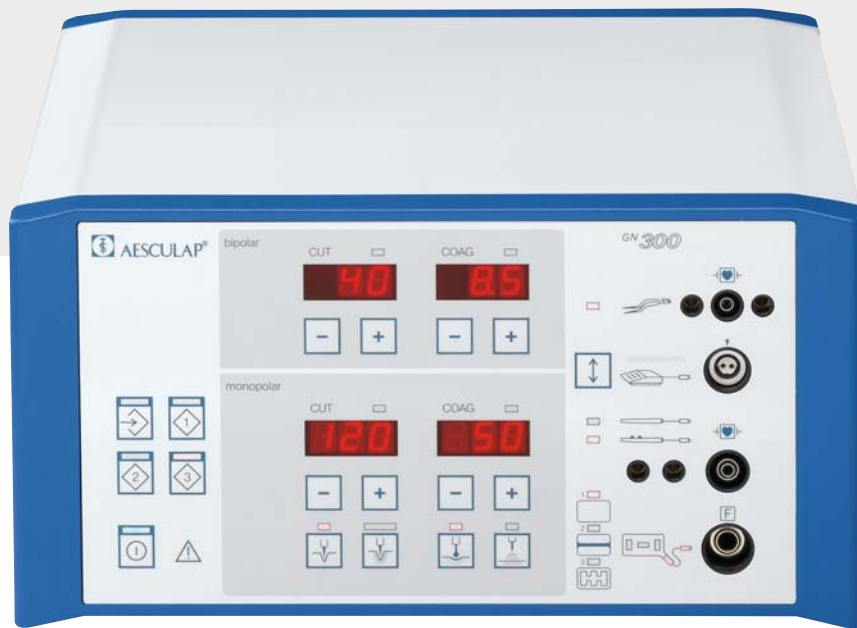
Features

- Reproducible cut qualities through microprocessor control
- Problem-free incision through PSF (Power Start Funktion)
- Controlled coagulation procedure by means of Variopedal
- Spray coagulation for effective surface coagulation
- Powerful bipolar coagulation with integrated power range for micro-coagulation
- Easy operation through individual memory keys
- Continuous monitoring of neutral electrode

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Features

The Nelson BiBox acts as an adapter, control and distributor unit for operating Aesculap bipolar hand switches with HF generators from other manufacturers (e.g. Erbe, Valleylab, Martin). The BiBox converts the Aesculap hand switch signal into a foot switch signal that the different HF generator can recognize. In addition, the single bipolar output on the HF device is distributed across the double bipolar HF connections of the Nelson BiBox, making two bipolar connections available with the Nelson BiBox (e.g. for a bipolar clamp and a bipolar scissor).

Using the BipoJet hand switches with other manufacturers' HF generators avoids having to use a foot pedal and thus reduces stumbling.

You can double the potential of your HF generator by connecting two bipolar instruments to one bipolar output!

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220 mm, 8²/₃"

200 mm, 7⁷/₈"

SUSI
SINGLE USE SURGICAL INSTRUMENTS

- Standard coagulation forceps ■ Standard Koagulationspinzetten
- Pincès à coagulation standard ■ Pinzas de coagulación de standard
- Pinze per coagulazione standard

- Aesculap tab connector - Aesculap Flachstecker - Fiche plate Aesculap - Enchufe plano Aesculap - Innesto piatto Aesculap

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Bipolar coagulation forceps with Sintram tips do not stick to coagulated vessels.

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Features

- Slimline, petite shape for delicate work that won't tire you out.
- Forceps tips with rounded, polished surfaces to reduce tissue adhesion.
- Parallel guidance guarantees exact contact of forceps tips.
- Sturdy insulating cuff made of high-grade plastic joins the handle ends.
- Smooth, self-cleaning plug connection ensures reliable, electrical contact.

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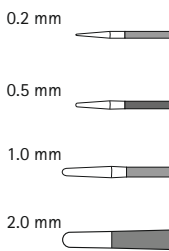
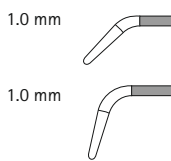
Features

- Slimline, petite shape for delicate work that won't tire you out.
- Forceps tips are rounded but the gripping surfaces are only finely worked to ensure good gripping of tissue.
- Robust, impact-resistant insulation coating.
- Finger recesses guarantee precise handling, particularly for deep operations.
- Smooth, self-cleaning plug connection ensures reliable, electrical contact.

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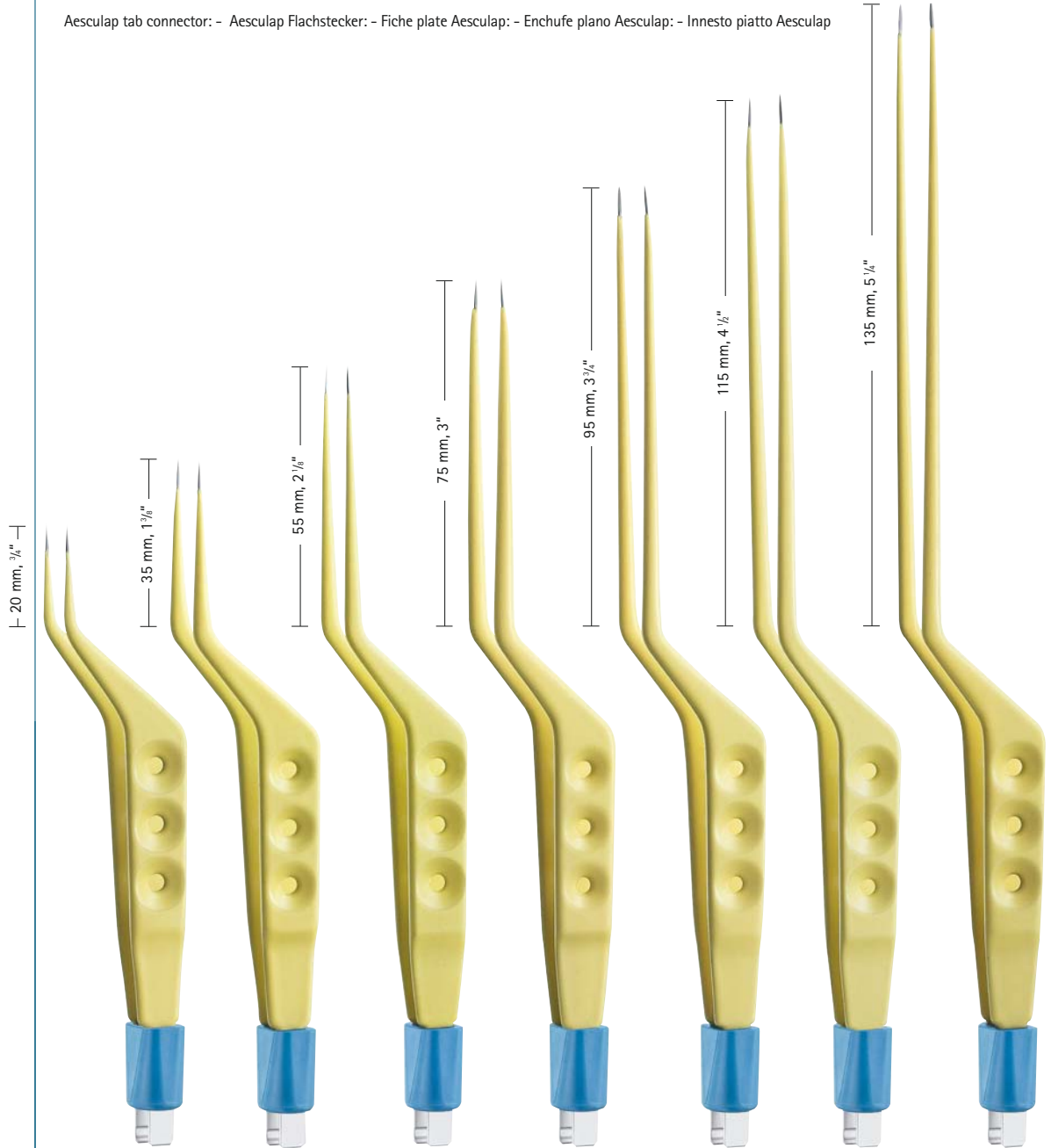
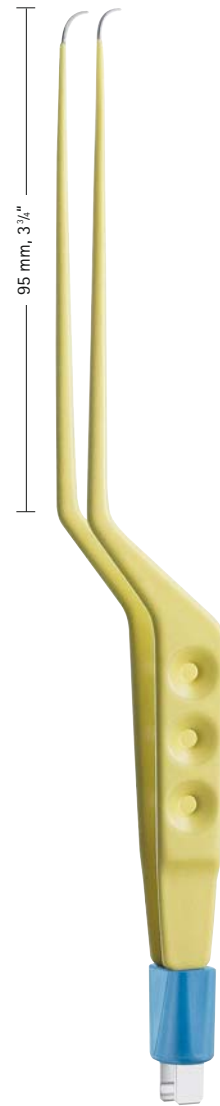
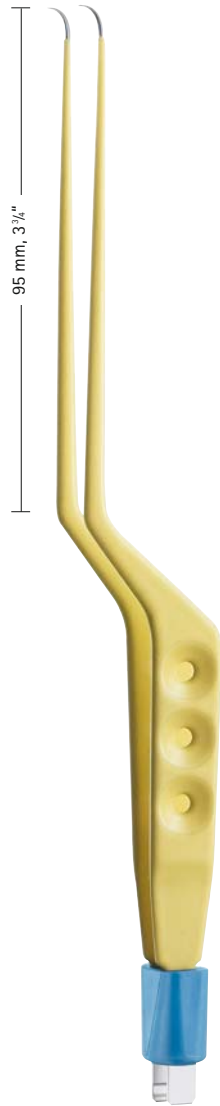
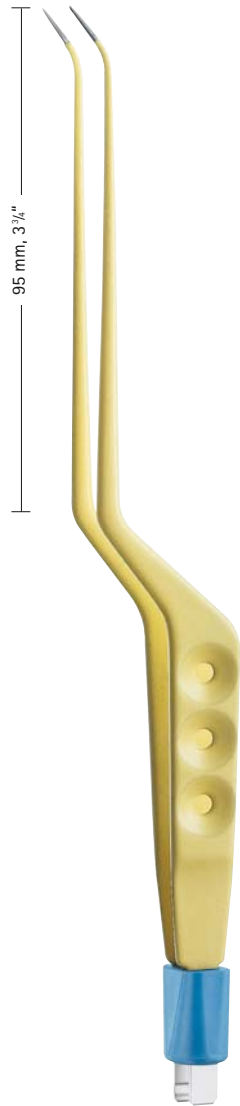
215 mm, 8 1/2"

215 mm, 8 1/2"

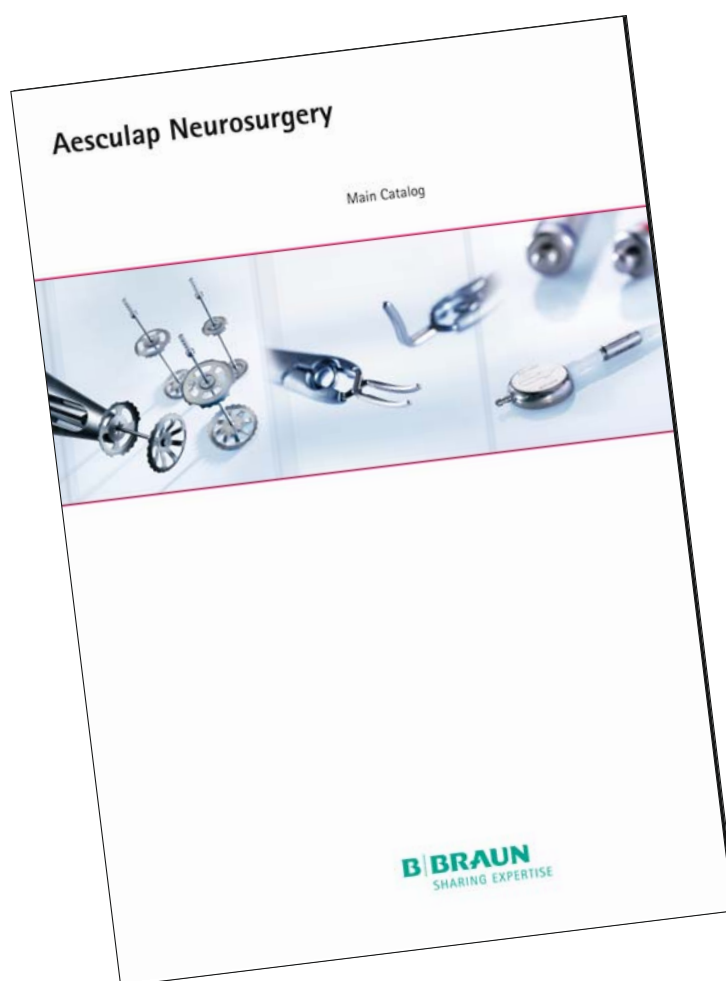
For more information see our Neuro Catalog C20102

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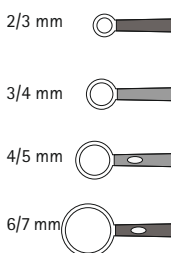
With the ring coagulation forceps, it is possible to perform resections of gliomas, meningiomas and acoustic neuromas.

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Coagulation forceps

for secure fixation and coagulation at the operation site, especially in the region of the pituitary

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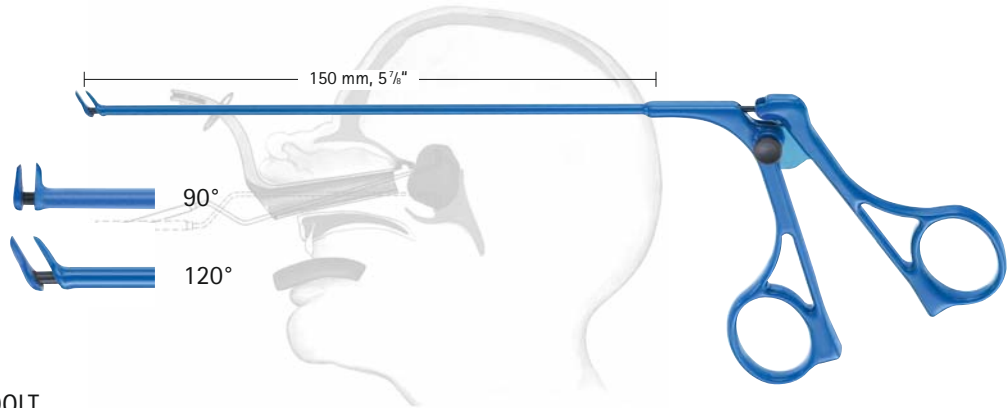
The JET irrigation system transforms an irrigation coagulation forceps into a multifunctional instrument.

The use of JET irrigation without coagulation makes it possible to wash away blood (for example insubarachnoidal haemorrhages) for exposing fine anatomical structures and enables subsequent coagulation without changing the instrument. In conjunction with drip irrigations, coagulation can be carried out gently and without tissue adhesion.

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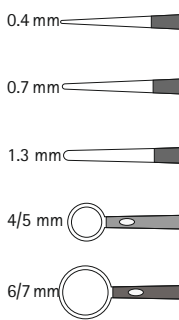
Cutting and coagulation in one step
– MORE TIME

Bipolar technology – MORE SAFETY

- The current flows through the tissue to be coagulated.
- No short circuits between the cutting edges.
- No colateral coagulation.
- Insulation up to the tip.

New instrument design – MORE SENSIBILITY

- New ergonomic handle.

Solid ceramic blade – MORE EFFICIENCY

- Scissors blades made of ceramics and instrument steel.
- Safe insulation. Thanks to the wear resistant ceramic inlay.
- Self-sharpening effect.
- Long service life.

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Complete ceramic insulation, coagulation only takes place between the jaws.

Ceramic teeth improve the grasping properties in the jaw.

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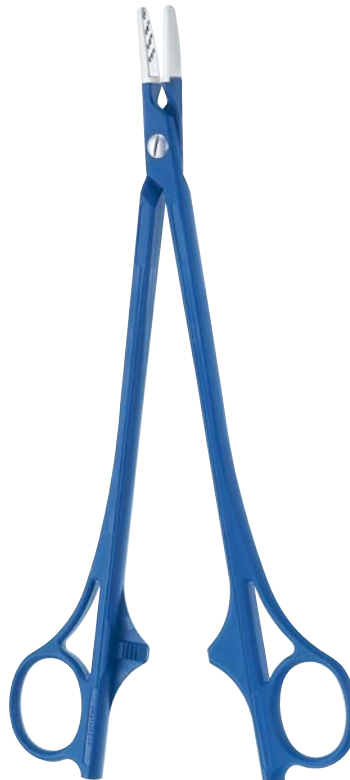
For older Aesculap
Generators

GN085
5 m

Aesculap
Bowa
Lamidey
Söring
Valleylab

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
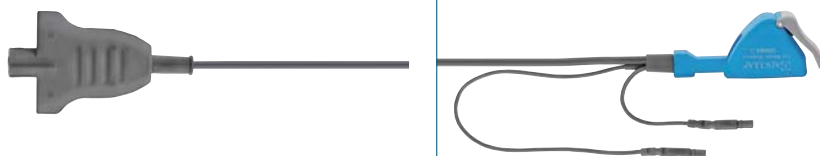
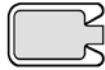
GK331
4 m

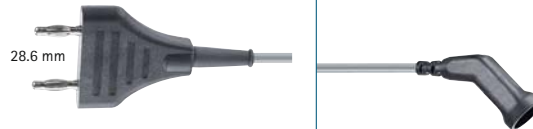
Aesculap
BOWA
Lamidey
Söring
Valleylab

Aesculap
- Round pin connector
- Rundstiftstecker - fiche ronde
- enchufe rondo - innesto perno roto

GK332
4 m

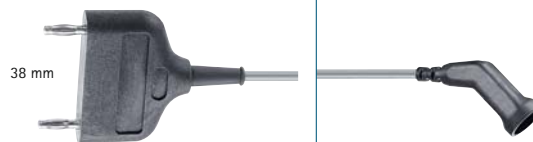Codman
CMC II
CMC III

Aesculap
- Round pin connector
- Rundstiftstecker - fiche ronde
- enchufe rondo - innesto perno roto

GK342
4 m

Erbe
Storz

Aesculap
- Tab connector - Flachstecker
- fiche plate - enchufe plano
- innesto piatto

GN140
4 m

Aesculap
ACMI
Berchtold

Aesculap
- Tab connector - Flachstecker
- fiche plate - enchufe plano
- innesto piatto

GN141
4 m

Martin
Wolf

Aesculap
- Round pin connector
- Rundstiftstecker - fiche ronde
- enchufe rondo - innesto perno roto

GN075
4 m

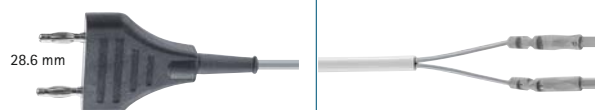
Aesculap
Martin
Berchtold

Aesculap Landolt
Coagulation forceps for hypophysectomy
Koagulationszangen für Hypophysektomie
Pinces à coagulation pour hypophysectomie
Pinzas de coagulación para hipofisectomía
Pinze per coagulazione per ipofisectomia

GK195
4 m

Aesculap
BOWA
Lamidey
Söring
Valleylab

Aesculap Landolt
Coagulation forceps for hypophysectomy
Koagulationszangen für Hypophysektomie
Pinces à coagulation pour hypophysectomie
Pinzas de coagulación para hipofisectomía
Pinze per coagulazione per ipofisectomia

	Equipment side Geräteseite Côté appareil Lado del aparato Lato apparecchi	Instrument side Instrumentenseite Côté instrument Lado del instrumento Lato strumento	
GN145 5 m	ERBE Int.		
GN146 5 m	Codman CMC II CMC III		
GK194 4 m	Aesculap BOWA Lamidey Söring Valleylab		 <ul style="list-style-type: none"> - Euro tab connector - Euro-Flachstecker - Fiche plate Euro - Enchufe plano Euro - Innesto piatto Euro
GN076 4 m	Aesculap ACMI Berchtold Martin Wolf		 <ul style="list-style-type: none"> - Euro tab connector - Euro-Flachstecker - Fiche plate Euro - Enchufe plano Euro - Innesto piatto Euro
GK185 4 m	Erbe Storz		 <ul style="list-style-type: none"> - Euro tab connector - Euro-Flachstecker - Fiche plate Euro - Enchufe plano Euro - Innesto piatto Euro
GK217 4 m	Codman		 <ul style="list-style-type: none"> Aesculap - Tab connector - Flachstecker - fiche plate - enchufe plano - innesto piatto

- Bipolar connecting cables
- Bipolare Anschlusskabel
- Câbles de raccord bipolaires
- Cables de conexión bipolares
- Cavo di collegamento bipolare

Equipment side
Geräteseite
Côté appareil
Lado del aparato
Lato apparecchi

Instrument side
Instrumentenseite
Côté instrument
Lado del instrumento
Lato strumento

GK280
4 m

ERBE Int.

Ⓜ Aesculap
- Tab connector - Flachstecker
- fiche plate - enchufe plano
- innesto piatto

GK281
4 m

Aesculap
BOWA
Lamidey
Söring
Valleylab

Ⓜ Aesculap
- Tab connector - Flachstecker
- fiche plate - enchufe plano
- innesto piatto

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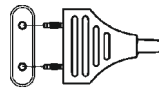
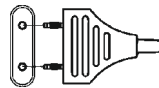
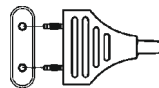
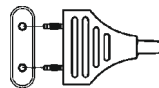
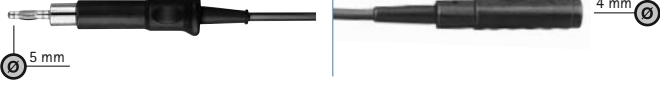
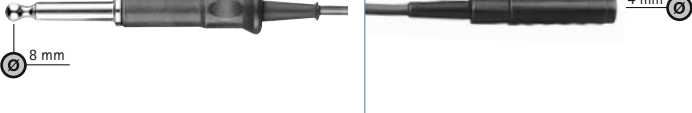
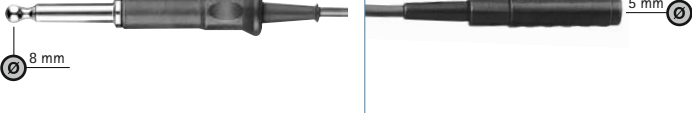
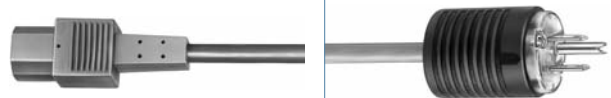
- Switchable lock
- Toggle switch mechanism

New jaw mechanism

- No protruding jaw hinges
- No getting caught of tissue and suture materials
- Using HF current

"One-Click" principle

- Single-touch mounting and demounting
- Effortless handling

PEEK insulation

- PEEK insulation of handle and shaft
- Improved sterilization resistance
- Guaranteed biocompatibility

Cleaning and sterilization

- All instruments separable into 4 components
- Irrigation port replaces the HF connector for cleaning a mounted instrument (incl. handle)
- Sterilization possible with the instrument mounted

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For further information, see catalogue

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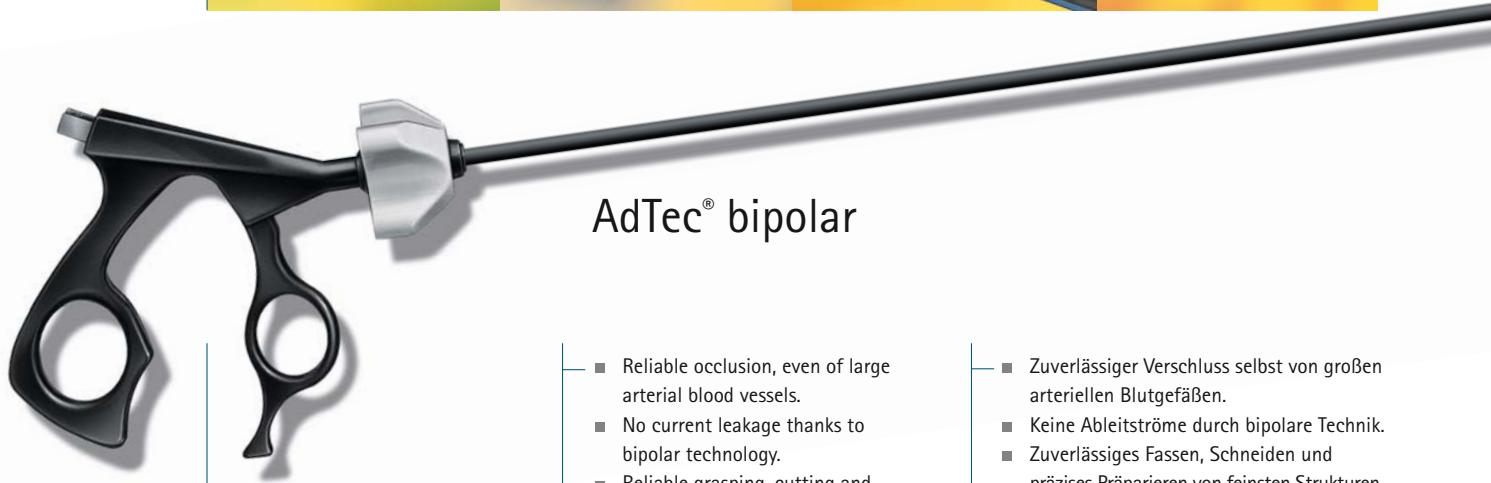
AdTec® bipolar

- Reliable occlusion, even of large arterial blood vessels.
- No current leakage thanks to bipolar technology.
- Reliable grasping, cutting and precise preparation of the finest structures.

High tactile feeling

The exact feeling of the touch is transmitted back to the user, letting him decide how firmly he wants to grasp the tissue.

Universal use

Can be used in all laparoscopic indications in gynaecology, general and vascular surgery.

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AdTec® mini

Functionality

The ergonomically designed components ensure easy and comfortable handling for the surgeon.

Reliability

The innovative mechanics enable precise grasping and controlled transfer of force alongside high durability.

Hygiene

The ease with which the instruments can be disassembled means modern hygiene requirements are met.

Service

Comprehensive customer support and cost-effective replacement of individual components are central to our philosophy.

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Art. No.	Page	Art. No.	Page	Art. No.	Page	Art. No.	Page
BC608R	32	GK084	72	GK180	81	GK321R	27
BC685R	63, 64	GK091	72	GK181	81	GK328	79
BC686R	64, 67	GK100	72	GK185	77	GK329	79
BC687R	63, 64	GK101	72	GK190	73	GK331	76
BC688R	64	GK107	72	GK191	73	GK332	76
BC689R	63, 64	GK110R	22	GK194	77	GK342	76
BC690R	64	GK112R	22	GK195	57	GK360R	93
BC691R	62	GK113R	22	GK217	77	GK361R	93
BC692R	62	GK114R	22	GK226	11, 15	GK362R	93
BC693R	62	GK115R	22	GK239	81	GK363R	93
BC694R	62	GK116R	22	GK245	81	GK364R	93
BC695R	62	GK117R	22	GK246	81	GK365R	93
BC696R	62	GK118R	22	GK252Y	20	GK366R	93
BH687R	67	GK121R	23	GK261	74	GK370P	91
BH689R	67	GK125R	24	GK266	74	GK372R	90
		GK127R	24	GK267	74	GK373R	91
FF380R	94	GK128R	24	GK280	78	GK383R	91
		GK129R	24	GK281	78	GK384R	91
GK005R	30	GK131R	25	GK282	78	GK385R	91
GK007R	30	GK132R	25	GK290SU	79	GK386R	91
GK009R	31	GK133R	25	GK291SU	79	GK393R	91
GK010R	30	GK140R	22	GK292SU	79	GK394R	91
GK013R	30	GK144R	22	GK299	23	GK395R	91
GK015R	31	GK145R	22	GK301R	27	GK397R	25
GK016R	31	GK146R	22	GK303R	27	GK398R	25
GK033R	31	GK155R	24	GK307R	27	GK424	25
GK080	72	GK164R	21	GK310R	27	GK432R	92
GK081	72	GK165R	21	GK315R	27	GK433R	92
GK083	72	GK174R	23	GK320R	27	GK434R	92

Art. No.	Page	Art. No.	Page	Art. No.	Page	Art. No.	Page
GK435R	92	GK680R	27	GK771R	52	GK846R	59
GK436R	92	GK685R	37	GK772R	52	GK847R	59
GK535	82	GK690R	38	GK773R	52	GK848R	59
GK550	33	GK700R	41	GK774R	52	GK870R	55
GK560R	57	GK701R	41	GK775R	52	GK871R	55, 59
GK561R	57	GK703R	41	GK777R	53	GK872R	55, 59
GK580R	57	GK711R	41	GK780R	53	GK881R	36
GK581R	57	GK713R	41	GK781R	53	GK882R	36
GK595R	42	GK719R	43	GK785R	53	GK883R	36
GK596R	42	GK720R	38	GK788R	52	GK884R	36
GK597R	42	GK722R	38	GK789R	52	GK885R	36
GK604R	44	GK723R	38	GK790R	52	GK886R	36
GK605R	44	GK730R	38	GK791R	52	GK890P	70
GK606R	44	GK743R	55	GK793R	52	GK891R	37
GK609SU	40	GK746R	52	GK800R	56	GK892R	37
GK614R	45	GK747R	52	GK801R	56	GK893R	37
GK615R	45	GK748R	52	GK807R	40	GK894R	37
GK616R	45	GK749R	52	GK808R	40	GK895R	37
GK624R	44	GK760R	52	GK809R	40	GK896R	37
GK625R	44	GK761R	52	GK831R	55	GK899R	48
GK644R	45	GK762R	52	GK832R	55	GK900R	48
GK645R	45	GK763R	52	GK833R	55	GK910R	48
GK646R	45	GK764R	52	GK838R	59	GK920R	48
GK660R	39	GK765R	52	GK840R	59	GK929R	48
GK670R	39	GK766R	52	GK841R	59	GK930R	48
GK671R	39	GK767R	52	GK842R	59	GK940R	48
GK675R	39	GK768R	52	GK843R	59	GK950R	48
GK677R	39	GK769R	52	GK844R	59	GK959R	48
GK679R	39	GK770R	52	GK845R	59	GK960R	48

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Aesculap – a B. Braun company

The main product mark 'Aesculap' and the product marks Nelson, AdTec and BipoJet are registered marks of Aesculap AG.

Subject to technical changes. All rights reserved.
This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

Brochure No. C30481

1112/1.5/15

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Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany